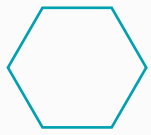


External Hexagon | Hybrid Implants



Versatility.



HEX-S | Solid

Enhanced Macrogeometry



- **Easy, Safe and Simplified installation (!)**
 - . Special Conical Drills have the same geometry of Hybrid Implants
 - . Only 2-3 Conical Drills to install Ø3.75 Hybrid Implants
 - . Does not require pilot drill, counter sink or screw tap
- **High Primary Stability, Balanced**
 - . Hybrid Macro Geometry
 - . Conical Apex | Parallel Body | Slightly Conical Coronal Flange
 - . Trapezoidal cutting threads | Torque Balance
- **Cortical Preservation Potential**
 - . Presence of Micro Threads up to platform flange
 - . Better stress distribution to cortical bone
 - . Higher coronal strength
- **Maximum Bone Contact**
 - . Combination of Hybrid Implants and Conical Drills
 - . Self Tapping
 - . 2 thread entrances
 - . Conical Solid apex | 3 cutting areas




HEX | Functional

Intellectual Property and latest development of Professor P-I Brånemark



- **Easy, Safe and Simplified installation (!)**
 - . Special Conical Drills have the same geometry of Hybrid Implants
 - . Only 2-3 Conical Drills to install Ø3.75 and 4.0 Hybrid Implants
 - . Does not require pilot drill, counter sink or screw tap
- **High Primary Stability, Balanced**
 - . Hybrid Macro Geometry
 - . Conical Apex | Parallel Body | Slightly Conical Coronal Flange
 - . Rounded single threads* | Torque Balance**
- **Short Implants from 6 mm**
 - . Ø3.75, 4.0 and 5.0
 - . Apex with 4 cutting areas
 - . Recommended for partial prosthesis

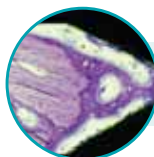


Superior interfacial neoformed bone (>BIC)



(!) See Surgical Sequence.

* Except for Ø5.0 - 2 thread entrances. ** Except HEX Ø4.0.

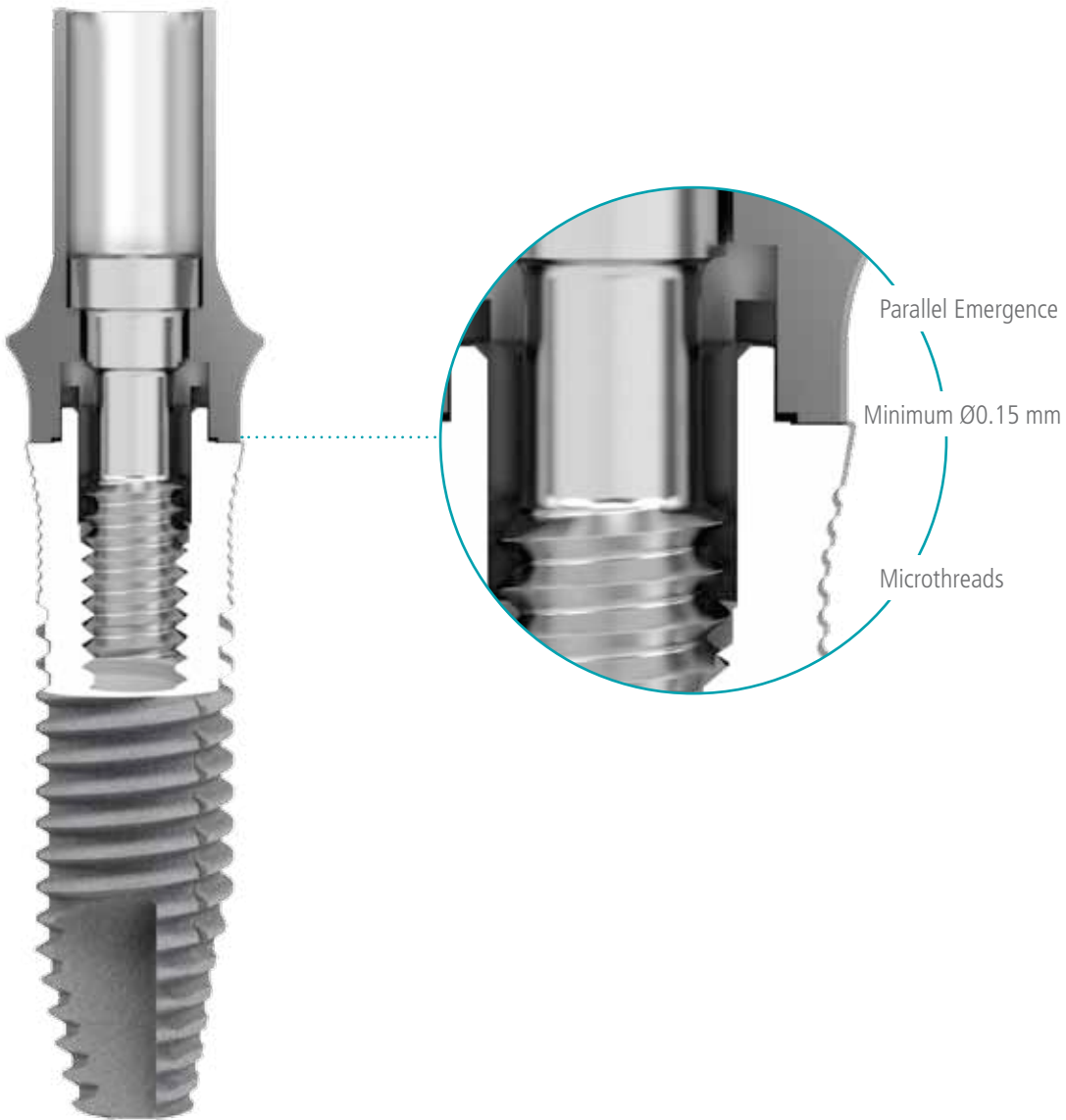Interface



- **Increased Biological Width**

- . Parallel Emergence Components
 - . Does not require removal of cortical bone tissue

- **Platform Switching for Enhanced Tissue Preservation**

- . Platform Ø5.1 has the same Hexagon of Platform 4.1, allowing use of 4.1 Components
- . Minimum Ø0.15 mm switching in all Platform diameters | 3.5 – 4.1 – 5.1

- **Compatibility**

- . Original Platform, Hexagon and Components Dimensions for Ø4.1

- **Mountless Installation**

- . Insertion Drivers with esthetic and dimensional references
- . Same Driver for manual, handpiece and wrench installation

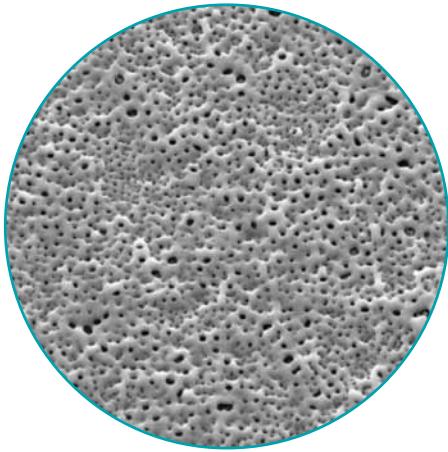
- **Versatile Interface**

- . Ideal for total and partial prosthesis
- . Easy prosthetic maintenance



Surfaces

P-I surfaces are modern and exhibit abundant Osseointegration properties



Widely Documented

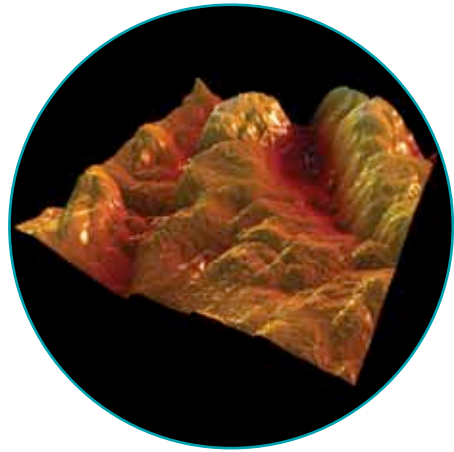
- Developed in the Department of Biomaterials – University of Gothenburg - Sweden and documented in many studies by some of the most important scientists in the field of implant surfaces.

Evolution of a Modern Surface

- A patented evolution of TiUnite® featuring significantly lower micro roughness, the Ospol® Surface is oxidized and incorporates Calcium Ions (Ca^{+2}) and presents similar results when compared to moderately rough surfaces.

Better Long Term Perspective

- Ospol® Surface represents a better hypothesis of improving long term success and longevity of Implants being less prone to biofilm adhesion (Periimplantitis), in clinical use since 2004.



Advanced Technology

- The Micro+Nano Surface is exclusively obtained by subtraction methods, controlled microblasting and Ions bombardment technology.




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

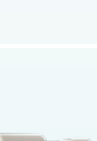




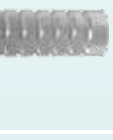

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
Minimally Rough and Nano Structured

- Exhibiting complex minimally rough micro structures and high density of nano features, designed for efficiency during healing periods, especially early ones, the Micro+Nano Surface is documented in international studies by worldwide experts in the Osseointegration field.

New Bone Areas


- A complete solution to address a wide range of clinical cases, the Micro+Nano Surface showed slightly increased bone areas in the 3 week period when compared to Ospol® Surface.

Multiple Screw Retained Single | Multiple Cemented Retained Single | Multiple Screw or Cemented Retained CAD/CAM* Overdenture

<p>Cylinder Scan Body</p> 						
<p>Healing Cap</p> 						
<p>Abutment</p> 						
<p>Healing</p> 						
<p>Implant</p> 						


* Please refer to the CAD/CAM Solution including 

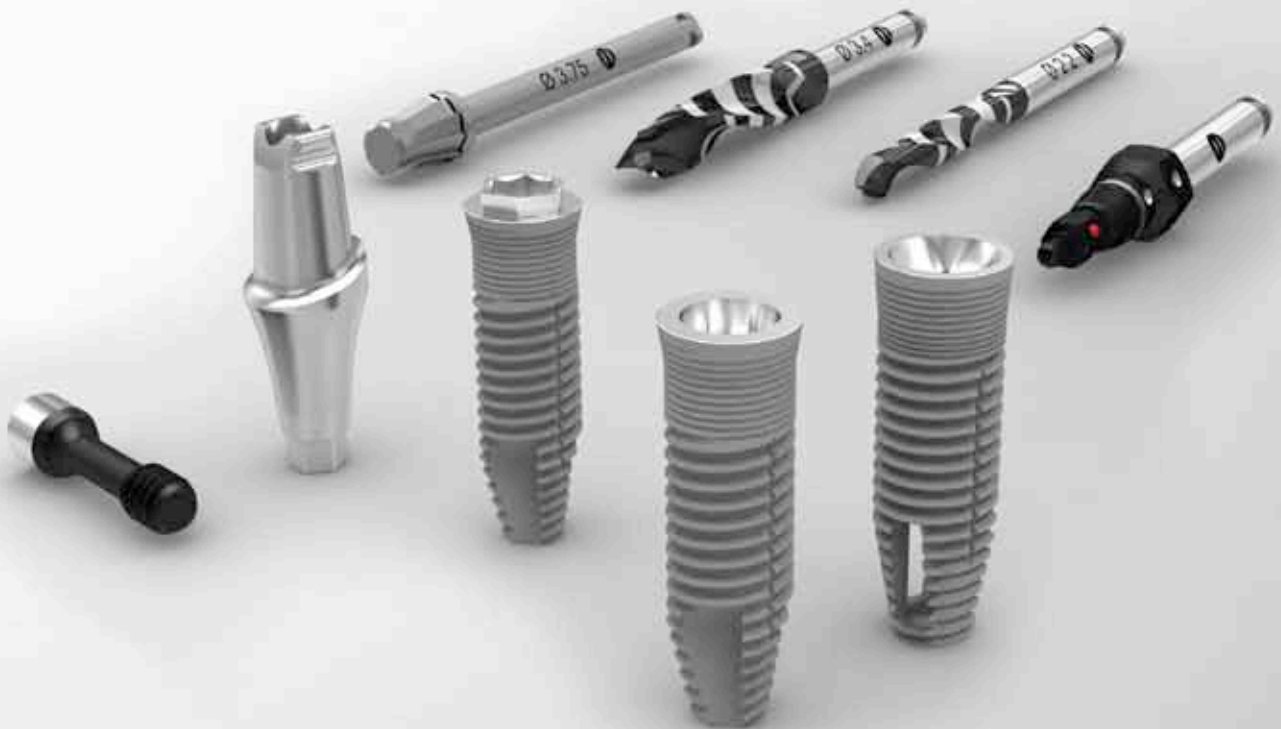Prosthetic Solutions



DATA SHEET HEX-005-ENG-2016-06-16



 Zimmer Dental Sweden AB
Industrivägen 4, SE-433 61
Sävedalen - Sweden

This material contains data from internal files and independent published scientific and clinical articles.
Please refer to www.pibranemark.com for additional product information.
Zimmer Biomet, Zfx™, P-I, Ospol®, Revitalize™ and www.pibranemark.com are Zimmer trademarks and domain.
Other trademarks are for reference only and belong to other companies.
(!) Some products may not be available in your region. Please check availability.